# 3-DMEd® LEARNING THROUGH SIMULATION








ECMO Simulation Kit User Manual

#### **Setup Instructions**

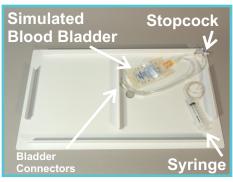


1. Identify parts of ECMO - Simulation Kit



2. Turn stopcock to position shown. Flip platform over.



3. Pull bladder connectors through hole so they sit on top of the platform.

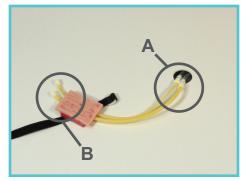

4. Connect the open ECMO pad connectors to the bladder connectors (A). Remove caps from the opposite ECMO pad ends(B).




5. Use the syringe to fill the bladder with simulated blood by connecting to an ECMO pad connector. Once full, replace connector caps.



6. Turn stopcock 90 degrees counterclockwise and return syringe to microtube on underside of platform. Syringe will allow for pulse and pressure.



7. Velcro® ECMO skin pad around the neck of manikin.

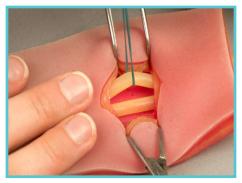

8. Vessels exposed in ECMO skin pad. We recommend using a French 10 cannula to cannulate the artery and a French 14 cannula for the vein.

#### Warranty

In order to provide warranty or future service, please contact us via e-mail at: <a href="mailto:support@3-Dmed.com">support@3-Dmed.com</a> prior to taking any action. This will help expedite the process. Problems can be resolved by e-mail or phone.

- **3-DMEd**®, a division of 3-D Technical Services Co., warrants this Product against defects, material, or workmanship subject to the following conditions:
- 1. MONITOR and CAMERA: These items are covered by their manufacturer's warranty for a period of one year. 3-Dmed® will act as liaison in the case of your claim, but final determination of coverage is the manufacturer's decision.
- 2. LABOR: For a period of 90 days from the date of purchase, if **3-Dmed**® determines the Product (part) to be defective, we will repair or replace the Product (part) at no charge. After the warranty period, we will provide an estimate for labor charges upon request.
- 3. PARTS: **3-Dmed** will replace defective parts with new or rebuilt replacements for a period of 90 days. After the warranty period, parts are available through **3-Dmed** and pricing will be provided upon request.

TO OBTAIN WARRANTY SERVICE: YOU MUST OBTAIN A RETURN AUTHORIZATION NUMBER via phone or e-mail BEFORE sending the unit back. The Product must be sent in its original packaging, or packaging affording an equal degree of protection, to 3-Dmed® with insurance for its full value. Uninsured parcels are sent at YOUR RISK for loss or damage.

This warranty does not cover user cosmetic damage or damage due to acts of God, accident, misuse, abuse, negligence, or modification to any part of the Product. This warranty does not cover damage due to improper operation or maintenance, connection to improper voltage supply, or attempted repair by anyone other than a trained technician at 3-Dmed. This warranty is invalid if the factory applied serial number has been altered or removed from the Product.

#### **Laparoscopic Task Inserts**







Soft Tissue Suture Pad



Tray of Organs



Skill Building Task 1 Loops and Wire



Skill Building Task 2 Pea on a Peg



Skill Building Task 3 Wire Chaser



Skill Building Task 4
Post and Sleeve



Practic@Board for FLS Skills



Vaginal Cuff and Holder



**Bowel Sections** 



#### More Products from 3-DMEd®




**Minimally Invasive Training Systems** 



**ENA Suture Kit** 



Skin Pad Holder

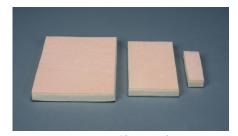

3-Layer Skin Pads




**Mock Medical Instruments** 



Adjustable Height MITS
Work Station



Lifelike Biotissue

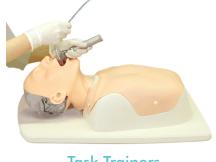


**Task Trainers** 




**Organ Crossword Game** 



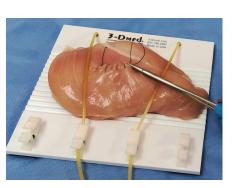

**Tissue Mounting Board** 

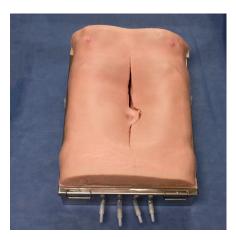

**AAA Abdomen Simulator** 

### Contact Us

#### Headquarters

255 Industrial Drive Franklin, OH 45005, USA +1 +(937) 746-2901

#### Sales

Jeff Worsinger +1 +(612) 201-6653 sales@3-Dmed.com

## 3-DMEd.com







in