

Instruction Manual for Oral Care Model

This is the simulation model for the training in the oral health. We would be grateful if this model assists you in the oral care practice as an aid.

Contents

Soft lower face, Transparent mask, Artificial tooth, Dental plaque, Stand, Mirror, Brush, Case

Before use

1. Don't use the colored fluid, otherwise the model would be stained and it could not be cleaned off. Use the water.
2. Don't mark with any pens like felt pen by same token.
3. Dry the model after use, otherwise it might cause mold.
4. We have the soft lower face available as it is consumable.
5. The packing material in the case might be yellowing occasionally, but it is not determination in the quality of product.
6. Could you kindly contact us or our distributor when you find any defects with this product.

Way of use

1. Remove the black knob in the back of the soft lower face by turning it counterclockwise, when using, and put it on the stand. Then put the black rubber sheet in between the transparent mask and the stand.
2. Adjust the horizontal angle of the face, turning the black knob clockwise or counterclockwise.

It could be changed in the seven steps lengthwise.

Be careful not to turn the knob too tight.



Kyoto Kagaku Co.,Ltd

15 Kitanechoya-cho Fushimi-ku Kyoto 612-8388 JAPAN

Phone +81-75-605-2510 / Fax +81-75-605-2519